



STAND-ALONE SURGICAL CART

SPECIALIST CLINICAL PERFORMANCE

STAND-ALONE SURGICAL CART

SPECIALIST CLINICAL PERFORMANCE



BU Medical Equipment
Sede Legale Ed Amministrativa
Headquarters
CEFLA s.c. - Via Selice Provinciale, 23/a
40026 Imola (BO) - Italy
Tel. +39 0542 653111
Fax +39 0542 653344

Stabilimento
Plant
CEFLA s.c. - Via Bilocca, 14/C
40026 Imola (BO) - Italy
Tel. +39 0542 653441
Fax +39 0542 653601

Due to our policy of constant technological upgrading, technical specifications may be subject to change without prior notice. According to the standards in force, in extra-EU areas the availability and specifications of some products and/or characteristics may vary. Please contact your local distributor for further information.

CASTELLINI.COM

03/2021 CCCAS9B211S00



CASTELLINI.COM

AN INTEGRATED SPECIALIST TREATMENT CENTRE

The Stand-Alone Surgical Cart has been added to the Castellini range to give dentists the potential to practice implantology and endodontics at the highest level, without being bound to the purchase of a chair. As an option, this solution can be purchased individually in a plug & play version. Where multiple connection boxes are located in different rooms within the surgery, all users need to do is connect the cart in the room where they're working.

A cart to free up your potential



FREEDOM TO MANAGE SURGERY SPACE EFFECTIVELY

The Stand-Alone Surgical Cart can be configured with integrated instruments, programmes and functions, ensuring endodontic and implantology tasks are performed smoothly. What's more, the version that quick-couples to a floor-mounted connections box via a special socket lets dentists move from one surgery room to another without needing more than one cart. With this solution, room-to-room transfers require no work on the part of a technician and initial investment and subsequent maintenance costs are lower. If you have a patient chair for orthodontic work without a dentist's module, the cart opens up the possibility of performing specialist treatment.



TOUCH SCREEN CONTROL PANEL

Two Full Touch control panels are available: Clinic and Multimedia. Both have a 7" capacitive multitouch screen and provide plenty of scope for personalisation of programmes, instruments and displayed data. With the Multimedia version (optional), users can view videos, photos and X-ray images loaded via the USB port. The software can be configured for traditional endodontics or reciprocating-mode instruments. For implant procedures, a peristaltic pump and torque curves are available.

To help streamline workflows, instrument housings have been designed to minimise the risk of accidentally dropping the instrument during treatment.



PLUG&PLAY CONNECTOR

When the cart needs to be moved from one room to another, the plug&play connector can be detached quickly. Designed to be hard-wearing, the plug provides a secure connection and the socket is protected against dust and other risks by a flap.



FIXED BOX

The standard version of the Stand-Alone Surgical Cart has its own connections box. The connector has extensions measuring 3, 4, 5 and 6 metres in length, allowing optimal flexibility in each room. This set-up provides plenty of operating space and lets dentists carry out specialist clinical treatment.



WIRELESS FOOT CONTROL

Makes it easier to move equipment, frees up more space and minimises hindrance. What's more, the foot control has an array of key functions (e.g. instrument and peristaltic pump control) and reduces manual contact with the control panel.

CREATED TO MANAGE IMPLANT PROCEDURES



The cart features devices and software that allow smooth performance of implant surgery. For example, it includes a powerful induction micromotor, a peristaltic pump and a cutting-edge control panel that lets users quickly customise and save speed, torque and irrigation flow parameters. Moreover, further implantology features include the ability to record torque curves. Data can be exported in three different formats so it can be used as a support for subsequent analysis, for quick reference purposes or simply as documentation to be filed in the patient's records.




GOLDSPEED EVO R20-L CONTRA-ANGLE

Specific for surgery. Autoclavable and thermodisinfectable, it can be disassembled for thorough cleaning. With internal cooling, external spray and LED with integrated generator.




IMPLANTOR LED MICROMOTOR

Noiseless and ultra-light, the Implantor LED micromotor is vibration-free and characterised by extremely high torque. With suitable contra-angles, it can deliver up to 70 Ncm.

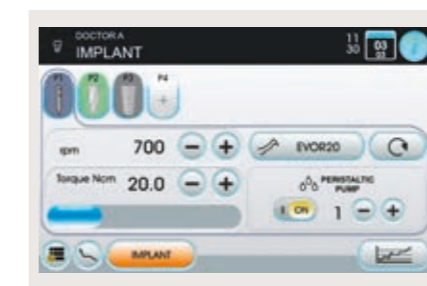

OPERATING PARAMETERS

The display shows only the information relevant to the clinical needs of the moment: all working parameters can be modified with ease at the touch of a finger, thereby leaving the dentist free to focus on the operation and the patient.

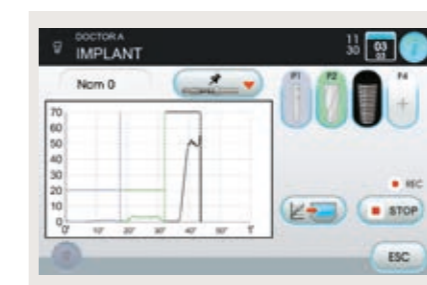

TORQUE CURVES

The dentist can also use an integrated function that records torque curves during surgery. Moreover, following treatment documentation is produced in an exportable format to keep a record of the delivered torque and implant tightening torque.

THE BEST FOR ENDODONTIC TECHNIQUES




With the Stand-Alone Surgical Cart, endodontic tasks and correlated treatment are assisted to ensure dentists receive valuable help. Thanks to precision control of the bur and the integrated database, the cart provides perfect support for endodontic specialists. Additionally, it provides them with highly sophisticated control parameters such as Autostop, Autoreverse and Autoforward. Perfect control of reciprocating mode is also provided for.




RECIPROCATING MODE

Together with the EVO E4 contra-angle, the control mode with alternating rotation allows for the use of RECIPROC®, RECIPROC BLUE® and WAVEONE GOLD® reciprocating files. The endodontic file brand names are not owned by Cefla or any of its subsidiaries.

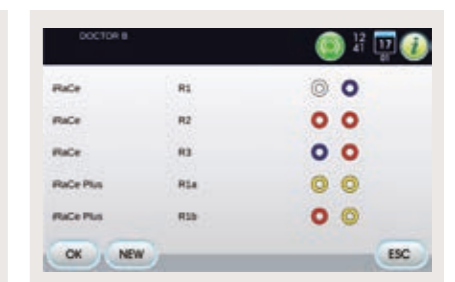

DATABASE

For all traditional endodontic procedures, the control panel is able to automatically set optimum parameters thanks to an inbuilt database of the most common endocanal files.

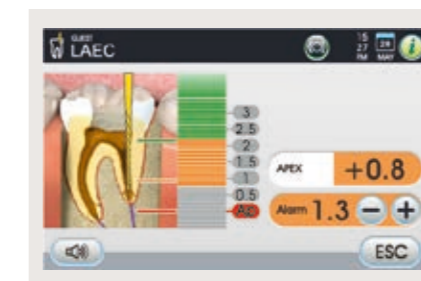

LAEC - INTEGRATED APEX LOCATOR

The device helps the dentist locate the root apex through an audible and visual signal. Combined with the EVO E4 contra-angle, it also integrates the APEX-STOP function which automatically switches the file off upon reaching a set safety threshold.

INSTRUMENTS WITH CUTTING-EDGE TECHNOLOGY

Built according to the highest technological standards, all instruments available with the Stand-Alone Surgical Cart are designed to meet the dentist's every need. Precise, reliable, light and easy to handle, these devices improve every aspect of implant surgery and conservative and endodontic dentistry. What's more, all their functions are controlled by the software on the Full Touch control panel.



PIEZOLIGHT 6

Available with fibre optic for LED lighting, the versatile and multifunctional ultrasound handpieces are suitable for prophylaxis, endodontics and periodontal procedures.

- Perfectly linear and controlled ultrasonic vibrations - free from parasitic lateral vibrations - with constantly self-regulated frequency according to load and tip conditions.
- Extensive power adjustment range.
- Self-regulating power based on the workload to prevent heating of handpiece and treated tissues.
- Coolant delivery adjustable from zero to max flow rate.
- Completely autoclavable.



SURGISON 2

Specifically designed for oral surgery, implantology and osteotomy, this handpiece is available with a wide range of accessories for any treatment requirements. Interchangeable with Piezolith 6 and compatible with Piezosteril 6 (following replacement of the tubing), the Surgison 2 handpiece is always automatically detected and piloted by the cart. Completely autoclavable, it provides optional irrigation with sterile fluids for complete asepsis of the operating field.



T-LED

The curing light can be used as a straight or 120° angled version. With an ultra-high power LED up to a maximum of 2,200 mW/cm² and a 430 - 490 nm emissions range, the T-LED has 7 pre-set programmes and an autoclavable light guide.



SYRINGE

Featuring a streamlined and easy-to-handle design, the syringe provides six functions. The handle and tip are autoclavable.



SILENT POWER TURBINES AND GOLDSPEED CONTRA-ANGLES

Castellini offers a complete set of turbines and contra-angles suitable for all fields of dentistry.

CONSTANT ATTENTION TO HYGIENE



TIME FLUSHING

To carry out fast, intensive flushing between patients, or immediately after switch-on in the morning, the Time Flushing function performs a complete water change by way of a timed feed fluid flow.



S.S.S. BOTTLE

The stand-alone supply system is equipped with a 1.8 litre capacity pressurised tank, easily accessible from outside to facilitate filling or replacement.



DETACHABLE INSTRUMENT TUBING

Removable and easily disinfected, the tubes make cleaning and replacement operations easier and faster.



GLASS DISPLAY FILM

Disposable film for the protection of assistant's keypad and Full Touch control panel.



SILICONE MAT

Easily removable and autoclavable, the mat is made of silicone for added safety.

A CONNECTED SURGERY



VIRTUAL ASSISTANT

Di.V.A. (Digital Virtual Assistant) is a virtual assistant that optimises the workflow in advanced dental practices. In addition to being viewable on a PC, Di.V.A. can be easily consulted on smartphones and tablets.


DASHBOARD

A simple dashboard lets the dentist monitor cart performance across all functions and analyse dental practice efficiency. In larger surgeries this is an excellent tool for managing significant workloads and planning maintenance.

TECHNICAL SPECIFICATIONS

STAND-ALONE SURGICAL CART	
HYGIENE SYSTEMS	
Separate Supply System	<input type="radio"/>
Time Flushing (quick wash cycle of the water circuit)	<input type="radio"/>
DENTIST'S MODULE	
Clinic 7" Full Touch display	<input checked="" type="radio"/>
Multimedia 7" Full Touch display	<input type="radio"/>
LED Implanter micromotor with f.o. (100 - 40,000 rpm, 5.3 Ncm)	<input checked="" type="radio"/>
Scaler (with or without f.o.)	<input type="radio"/>
Peristaltic pump kit	<input type="radio"/>
"Torque curves" implantology function	<input type="radio"/>
6-way syringe	<input checked="" type="radio"/>
Sixth instrument	<input type="radio"/>
Reciprocating mode module	<input type="radio"/>
Integrated Apex Locator	<input type="radio"/>
Surgison 2	<input type="radio"/>
Multi-function foot control	<input checked="" type="radio"/>
Pressure-operated foot control	<input type="radio"/>
Power Pedal foot control	<input type="radio"/>
Wireless multi-function foot control	<input type="radio"/>
Wireless pressure-operated foot control	<input type="radio"/>
Power Pedal foot control - wireless	<input type="radio"/>

● = as standard ○ = optional - = not available

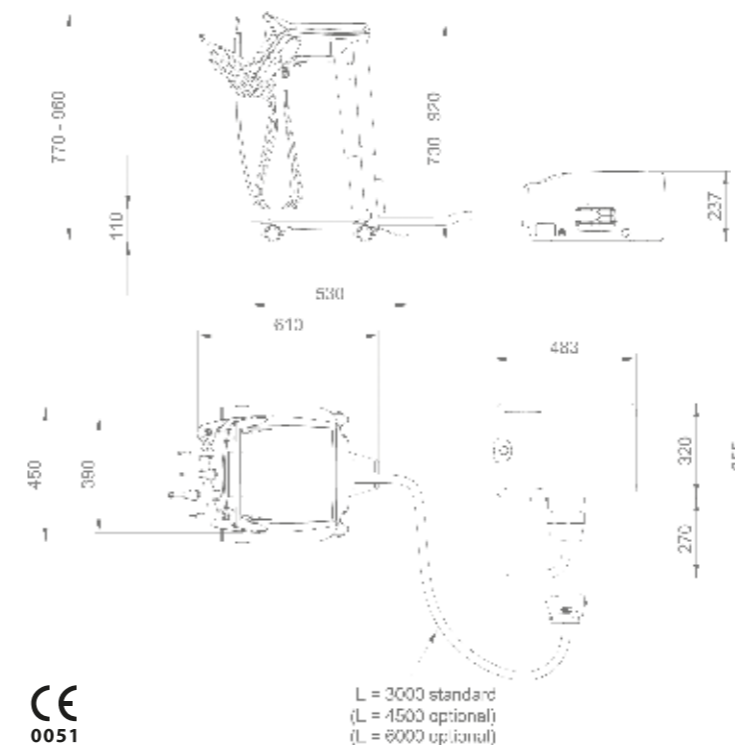

Measurements in drawings are in millimetres.